

PRODUCT DATA SHEET

E-FOAM

Highly-Effective Foam Moisturizing Cleaner



DESCRIPTION

A ready-to-use surgical pre-cleaner that removes heavy soils from surgical instruments, maintains moisture during transportation, and protects valuable instruments.

INTEND USAGE

Reprocessing of surgical instruments: precleaning and moisturizing

PROPERTIES

- Rapid sterilization, takes 2 minutes to kill enteropathogenic bacteria, pyogenic cocci, pathogenic yeast and common bacteria
- Durable bacteriostatic effect lasting more than **48** hours
- Long-lasting dense foam, keeps instruments moist for 24 hours, prevents corrosion
- Dissolves blood and fat, making cleaning instruments fast and effective
- Neutral pH allows safe use with most materials
- Non-toxic, all components are biodegradable and eco-friendly

Dacter Instatte Test							
Test Item	Colony Before Treatment	Contact Time	Bacteriostasis Result				
Escherichia coli ATCC8099	7.6*10 ⁴ cfu/ml	10min	99.10%				
Staphylococcus aureus ATCC6538	6.2*10 ⁴ cfu/ml	6.2*10 ⁴ cfu/ml 10min					
Pseudomonas aeruginosa ATCC 15442	6.5*10 ⁴ cfu/ml	10min	99.89%				
Candida albicans ATCC 10231	5.1*10 ⁴ cfu/ml	2min	99.99%				

Bacteriostatic Test

According to "Disinfection Technical Regulation" 2002 edition.

Dalian Lion Medical Industries Co., Ltd. No. 2-1, Qinggang Street, Free Trade Zone, Dalian, China T:+86-188-0428-5618 W: www.lionmedical.cn E:export@lionindustry.cn



Bactericidal Test

Test Item	Colony Before	Contact	Bacteriostasis				
Test ttelli	Treatment	Time	Result				
Escherichia coli ATCC8099	6.2*10 ⁴ cfu/ml	10min	99.05%				
Staphylococcus aureus ATCC6538	5.9*10 ⁴ cfu/ml 10min		99.43%				
Pseudomonas aeruginosa ATCC 15442	4.1*10 ⁴ cfu/ml	10min	99.34%				
Candida albicans ATCC 10231	6.0*10 ⁴ cfu/ml	10min	99.90%				

According to "Disinfection Technical Regulation" 2002 edition.

Corrosion Test								
Material	STS	Al	Cu	PVC	Rubber			
Corrosion rate	< 0.01	< 0.01	< 0.01	< 0.01	< 0.01			
mm/a	<0.01	~0.01	<0.01	<0.01	<0.01			

According to T/WSJD002-2019 "Hygienic Requirements For medical Cleaning Agents"

INSTRUCTIONS FOR USE

Ready-to-use with foam generating device

PHYSICAL AND CHEMICAL PROPERTIES

- Appearance: Transparent solution
- Composition: Bactericidal factor, surfactant, moisturizing, anti-corrosion factor, foam stabilizer
- pH: 7.0 ±0.5
- Aerobic bacterial: <10CFU/mL
- Coliform bacterial: <10CFU/mL
- SVHC: NA
- Shelf life: 2 years

TRANSPORTATION AND STORAGE CONDITIONS

- During transportation: handle with care, do not turn upside down.
- Store in cool, dry conditions in well-sealed receptacles.
- Protect from heat and direct sunlight.
- Recommended storage temperature: 10°C-30°C.

PACKAGING

- 1L / canister, 12 canisters/carton
- 5L / canister, 4 canisters/carton

STANDARD

Q/DLE 001

Dalian Lion Medical Industries Co., Ltd. No. 2-1, Qinggang Street, Free Trade Zone, Dalian, China T:+86-188-0428-5618 W: www.lionmedical.cn E:export@lionindustry.cn



